AQUISEL

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MEDICAL DEVICES MANUFACTURER FOR DISPOSABLE IN VITRO DIAGNOSTIC.

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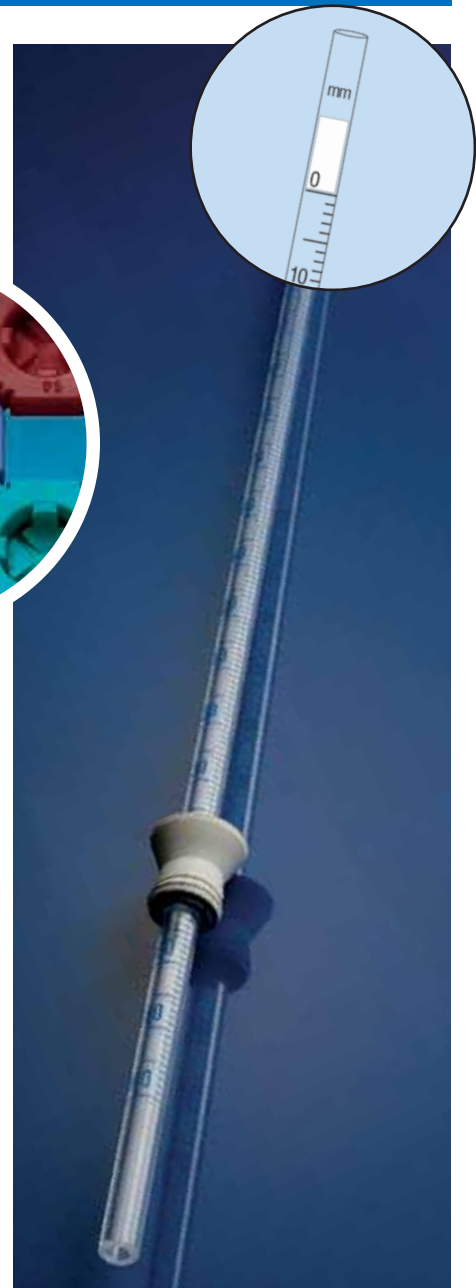
ICO-0011-0/02-2009

Pipettes for E.S.R. with zero levelling and automatic closure system for conventional tubes or AQUISEL Valve-Stopper.

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® **AQUISEL VALVE-STOPPER**

System available for sample manipulation without the need of uncorking the tube, maintaining it hermetically sealed and avoiding the risk of contamination.

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The AQUISEL system consist of a tube, valve-stopper and an appropriate reagent for specific use.

The AQUISEL tube is made of a high resistance transparent polypropylene. The valve-stopper is a very elastic material and **has a special cross-cut design which allows an automatic recover to its initial closed position, after insertion or extraction of the sample.**

The tube's reagent contents is offered through different possibilities identified by colours in accordance with International Standards.

It can be used by most of the current analytical procedures, even if they are manual, semi-automatic or automatic procedures.

It allows inter-hospital road and postal transport thanks to its hermetic seal and high resistance to uncorking due to vibrations, hits, cold, heat, etc.

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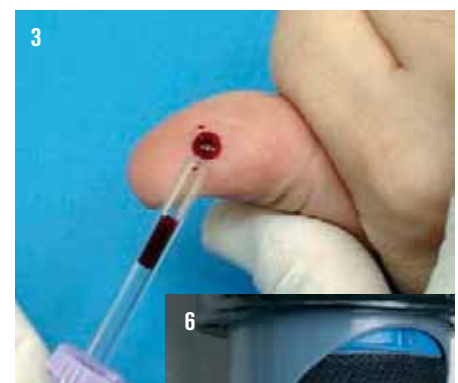
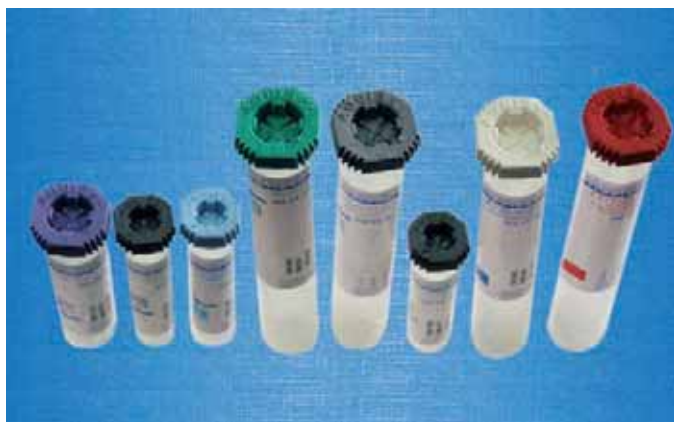
USE AND APPLICATIONS

1. Inserting (blood) samples with syringe (without needle).
2. Extraction and dosage with semi-automatic pipette.
3. Capillary blood extraction.
4. Capillary filling for microhematocrite.
5. Automatic sampling for haematology in primary tubes.
6. Sample road transport.
7. Centrifuge heads adaptable.

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There are different size and volume tubes with additives: anticoagulant, antiglycolitics, clotting accelerators or empty tubes.

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Suitable for paediatrics or small sample volumes.

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The tubes are presented in polystyrene racks of 100 units. Every 50 units are covered with a transparent lid for protection.

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Funnel adapter which can be used together with the primary AQUISEL tubes and allows sample blood transfer to secondary tubes, useful for capillary blood extractions in paediatrics, for low blood volumes, can be used in all auto-analysers for primary tube sampling, etc. All these procedures could be completed without uncorking the primary AQUISEL tubes. Once the CAP-GALET is removed, the tube remains sealed again for conservation or disposal.

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
AQUISEL PIPETTE

PIPETTES FOR ERYTHROCYTE SEDIMENTATION READ E.S.R. WESTERGREN METHOD.

Pipettes made in polystyrene with similar properties as glass ones.

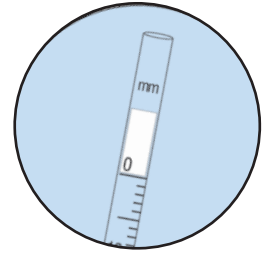The ZERO-CLOSURE system allows the pipette filling, preventing that the blood exceeds the zero point and also avoiding total or partial loss. Automatic sealing makes it a disposable product.

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AQUISEL Pipette A-2 without rubber.

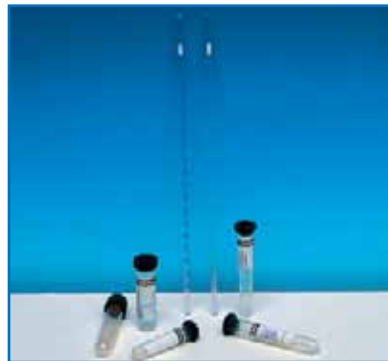
For any kind of tube use, even with AQUISEL or with a conventional stopper.

Suitable for mechanical or manual aspiration.

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AQUISEL Pipette P-2 with rubber.

For 75x13 tubes with AQUISEL or with a conventional stopper.

Pipette filling by pressure till the bottom of the tube.

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AQUISEL Pipette P-3 and P-4 MICRO systems

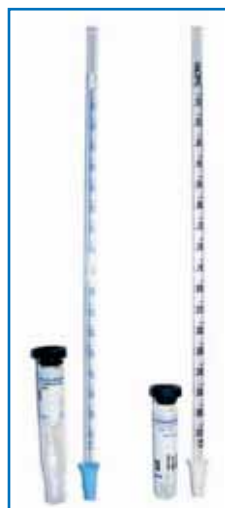
Set of AQUISEL pipettes and tubes. Tubes come with the valve-stopper which allows the pipette insertion without the need of uncorking.

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AQUISEL Pipettes A-2

Expanded polystyrene packaging of 500 units.

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AQUISEL Pipettes P-2

200 units cardboard package. Supplied in 800 units boxes.

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Set P-3. AQUISEL Pipette and tube

200 units cardboard package and 100 tubes rack made of expanded polystyrene, supplied in 600 sets boxes.

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Set P-4 Micro. AQUISEL Pipette and tube

100 pipettes cardboard package and 100 tubes rack made of expanded polystyrene, supplied in 400 sets box.

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Metallic Westergren stand

10 place autoclavable stand.

For any kind of AQUISEL pipettes with Ø 13 mm tubes and A-2 pipettes with Ø 16 mm tubes.

Easy and functional system that allows the total set disposal.

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1501226 para AQUISEL P-4 Micro (1000 uds/caja)

AQUISEL URINE CONTAINER

CONTAINER FOR URINE SAMPLES COLLECTION WITH AQUISEL VALVE-STOPPER.

The AQUISEL VALVE-STOPPER allows the urine manipulation without uncorking, eliminating the contamination risks, samples loss, spillage, odours, etc. All the AQUISEL urine container presentations will be supplied with the GALET accessory, which makes easy the sample dosage to a tube for sedimentation analysis, or the urine test strip insertion.

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Insertion of the GALET in the AQUISEL VALVE-STOPPER.

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Urine sample extraction through the GALET.

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Urine test strip insertion.

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Dosage through automatic or conventional pipette.

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